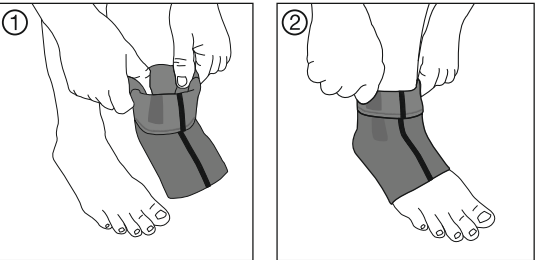
Cellacare® Malleo Classic

Sprunggelenkbandage
Bandage de la cheville
Ankle support

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en Instructions for Use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:

Cellacare® Malleo Classic is a knitted support with a compression effect which can improve proprioception. Silicone inserts on both sides provide a massaging effect. The support can be worn on the right or left ankle.

Product composition:

Cotton, elastane, elastodiene (natural rubber latex), polyamide, polyester, polyurethane, silicone

Intended purpose:

Support for compression and stabilisation of the ankle

Indications:

- Acute and chronic pain in the ankle joint
- Ligament injuries to the upper ankle (grade 1)
- Moderate chronic ligament instability
- Moderate local hypermobility
- Low grade osteoarthritis, activated osteoarthritis, arthritis
- Perimalleolar swelling, joint effusion, intra-articular irritation

Contra-indications:

- Known allergy and / or hypersensitivity to any of the product components

Side effects:

In very rare cases, skin irritations and / or allergies may occur.

Warnings and precautions:

- Do not use Cellacare® Malleo Classic unless directed by a health care professional.

The first time patients use the product they need to have a health care professional show them how to put it on correctly.

- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Malleo Classic shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before application of the product.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of circulatory and / or sensory disorders (e.g. with diabetes mellitus); arterial diseases (occlusive arterial disease); venous disease (phlebitis, venous thrombosis), impaired lymph circulation in the legs please contact your treating physician / seek medical advice before application of the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- Cellacare® Malleo Classic must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Malleo Classic is intended for use on a single patient only.
- The product's durability may be compromised by certain factors, e.g. objects with sharp edges.
- Cellacare® Malleo Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).

Cotton, elastane, elastodiene (natural rubber latex), polyamide, polyester, polyurethane, silicone

- This product contains natural rubber latex, which may cause allergic reactions.
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information:

Usage:

- Fold over the top of the Cellacare® Malleo Classic as far down as the silicone insert (Fig. 1).
- Grasp the Cellacare® Malleo Classic in the middle of the silicone inserts and pull it over your foot until the heel part is firmly in place (Fig. 2).
- Unfold the top of the Cellacare® Malleo Classic and pull it upwards, if necessary (Fig. 3). Ensure that the silicone inserts are sitting correctly either side of the ankle.

General information:

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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Care recommendations:

Wash the support at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® Malleo Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. If possible use a mesh bag.

Storage and transport conditions:

Cellacare® Malleo Classic must be kept away from light and moisture (preferably in the original packaging).

Disposal:

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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