Suprasorb® Liquacel Ag

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en Instructions for Use

Product Description

Suprasorb® Liquacel Ag Antimicrobial Wound Dressing is a soft, conformable non-woven fabric made from sodium carboxymethyl cellulose and strengthening cellulose fibers with antimicrobial silver. The silver in the wound dressing has an antimicrobial effect upon many of the various wound bacteria held within the dressing. Through the gel formation, debris and any bacteria found in the wound exudate can be retained inside the fibre dressing and removed when the dressing is changed. When dry, Suprasorb® Liquacel Ag can easily be cut to the size of the wound. Even when the dressing is moist, as a gelled fibre, its structure remains intact. The high vertical absorption of exudate into the dressing forms a gel which assists in maintaining a moist wound environment, supporting autolytic debridement, protects the wound edge and surrounding skin from maceration, thus supporting the healing process. The product can also be used under compression.

Intended use

Suprasorb® Liquacel Ag antimicrobial wound dressings can be used for the management of moderate to heavily exuding wounds.

Indications for use

Under the supervision of a healthcare professional, Suprasorb® Liquacel Ag antimicrobial wound dressings may be used for the management of:

- Partial thickness burns
- Leg ulcers, pressure ulcers and diabetic ulcers
- post-operative wounds (e.g. post-operative, wounds left to heal by secondary intent and donor/graft sites)
- Traumatic wounds (e.g. abrasions and lacerations)
- Wounds that are prone to bleeding, such as wounds that have been mechanically or surgically debrided and donor sites,

provided they are:

case of infected wounds, the treating physician will decide on the basis of the patient's general condition whether an occlusive secondary dressing can be applied.

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