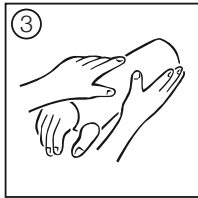
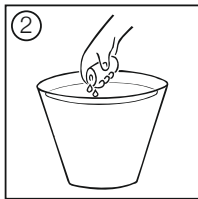
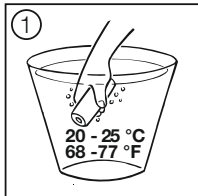
Cellona®

Gipsbinde · Bande plâtrée · Plaster of Paris Bandage · Venda enyesada · Ligadura gessada · Benda gessata · gipszwachtel · gipsforbinding · gipsbinda · Gipsbind · kipsiside · sádrové obinadlo · sadrové ovinadlo · gipszpólya · Bandaż gipsowy · Mavčni povoj · gipsani zavoj · ruženičecupoc · Paris alçı

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en Instructions for use

Product description and features:
Ready-to-dip Plaster of Paris bandage

Product composition:
Cotton gauze coated with a mixture of α- and β-calcium sulphate hemihydrate

Intended purpose:
Cellona® Plaster of Paris Bandages are intended to be used for moulding of immobilising casts on injured or fractured limbs and for moulding of splints or reinforcement of circular casts.

Contraindications:
Known allergy and / or hypersensitivity to any of the product components.

Side effects:
In very rare cases skin irritations and / or allergies may occur.

Warnings and precautions:

- Advise the patient against taking a bath with the cast, as Plaster of Paris materials are not resistant to moisture.
- Advise the patient that he/she should immediately disclose any sensation of numbness or pressure and must take care that no foreign bodies (e.g. sand) get under the cast, as this may lead to skin irritations.
- Cellona® P.o.P. products shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before application of the product.

Instructions for use:

1. Dip the bandage at an angle in water warmed to 20-25°C (approx. 1 sec. per metre of bandage).

Warning: The temperature of the water must not exceed 25°C, as this may lead to an increase in the setting temperature, which may cause burns in the patient.

2. Remove the bandage from the water and allow water to drip off. Then squeeze the bandage lightly to remove excess water.

3. Apply the Cellona® Plaster of Paris Bandages in a circular fashion without tension, ensuring that the layers overlap by approximately half. Then mould thoroughly to ensure a good bonding of layers. Take care that no pressure points are formed.

The setting time is 3-4 mins. Cellona® Plaster of Paris bandages are partially load-bearing after 30 minutes and fully load-bearing after 36-48 hrs. The cured Plaster of Paris can be removed with conventional instruments.

Storage and transport conditions:
Store in dry place under room conditions.

Disposal:
In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance - AVV). Packaging labelled as recyclable should be taken into the respective national recycling systems.

General instructions:

- For single use.
- In case of a serious incident, contact the manufacturer and the responsible health authorities.

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