Suprasorb® X

HydroBalance-Wundverband / -Tamponade
Pansement/Mêche HydroBalance

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en Instructions for Use

Product description and performance characteristics

Depending on the wound condition, Suprasorb® X HydroBalance wound dressings / packing ropes either donate moisture to the wound or absorb excess exudate. This creates a moist wound environment. Changing the dressing is atraumatic.

The material makes it easy to conform to the wound's particular shape and depth. The products are therefore especially suitable for areas of the body which are difficult to dress. The positive properties of the dressing resist adhesion to the wound or to the secondary dressing. The moist environment with a cooling effect is perceived by patients as pleasant and pain-relieving.

The products are sterile packed individually and sealed by irradiation.

Product composition
Suprasorb® X HydroBalance wound dressings / packing rope consist of cellulose and water.

Intended purpose
HydroBalance Bio-Cellulose wound dressings / packing rope aid wound healing by donating moisture and absorbing exudate from lightly to moderately exuding superficial or deep wounds.

-  Lightly to moderately exuding,
-  superficial or
-  deep wounds.

It appears as if the wound has the symptoms associated, the personal medical specialist should decide if to continue or no with the treatment:

- changes of color or odor of the herida
- stagnation of the process of cicatrization of the herida
- signs clinical of a infection

Indications
Suprasorb® X HydroBalance wound dressings / packing rope are suitable for:

- arterial and venous ulcers
- diabetic ulcers
- pressure ulcers
- superficial 2nd degree burns
- postoperative surgical wounds
- skin graft donor sites
- abrasions, lacerations

Contraindications
3rd degree burns
Known allergy and/or hypersensitivity to any of the product components

Side effects
In very rare cases skin irritations and/or allergies may occur.

Warnings and Precautions
Do not use if package is damaged or accidentally opened.

The continuation of treatment must be decided upon by a healthcare professional if any of the following symptoms should occur:

- change in colour and/or odour of the wound
- stagnant wound healing
- clinical signs of infection

Application instructions
Application of Suprasorb® X HydroBalance wound dressings / packing rope is performed by healthcare professionals.

1. Wound Preparation:
Cleanse the wound thoroughly. The surrounding skin should be clean and dry.

2. Apply the Wound Dressing:
The size of the dressing depends on the wound size. The dressing should overlap the wound by 2-3 cm.

- a) Before use, visually inspect the sterile barrier system for breaches to the integrity of the packaging.
- b) Open the sterile packaging and remove the dressing under aseptic conditions.
- c) If needed, cut the wound dressing and packing rope to size with sterile scissors. Leave the protective film in place on the dressing when cutting to size.
- d) Remove the protective film from both sides.
- e) Either side of the dressing may be applied to the wound.
- f) Place the wound dressing on the area to be treated and gently press or mould with the fingers.
- g) Secure with a suitable secondary dressing (Supasorb® F film dressing, Supasorb® P foam dressing). Protruding edges of the combined dressing are not a matter for concern.

When using the packing rope:
Both protective films must be removed from the product before it can be folded several times and loosely inserted into the wound. Do not completely fill the wound. Cover the packing rope with a suitable secondary dressing (Suprasorb® F film dressing, Suprasorb® P foam dressing).

3. Dressing change:
The intervals at which Suprasorb® X HydroBalance wound dressings / packing rope should be replaced and the duration of treatment are determined by the attending doctor depending on the condition of the wound. If used continuously, Suprasorb® X can be used for more than 30 days in individual cases. The dressing should be changed at least every 7 days, based on the condition of the wound.

Should the dressing or packing rope dry out, there must be adequate rehydration, with an exposure time of at least 10 minutes, before removal.

- a) Remove the secondary dressing.
- b1) Carefully lift one edge of the Suprasorb® X HydroBalance wound dressing, and then gently peel the dressing off the wound.
- b2) Gently pull out the Suprasorb® X HydroBalance packing rope from the wound.
- c) Do not reuse the removed wound dressing or packing rope.
- d) Cleanse the wound before applying a new wound dressing or packing rope.

4. Disposal
In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance - AVV). Recyclable packaging should be recycled according to the relevant national recycling systems.

General instructions
In case of a serious incident, contact the manufacturer and the responsible health authorities.

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