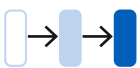
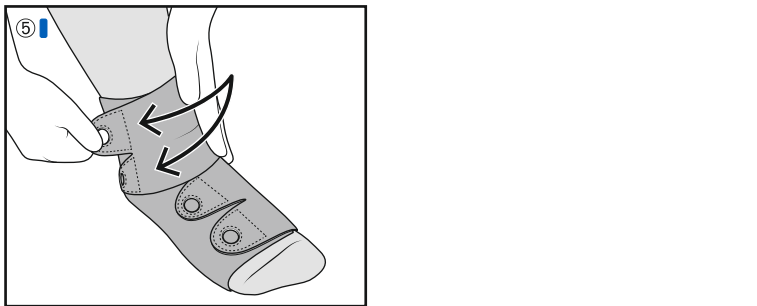
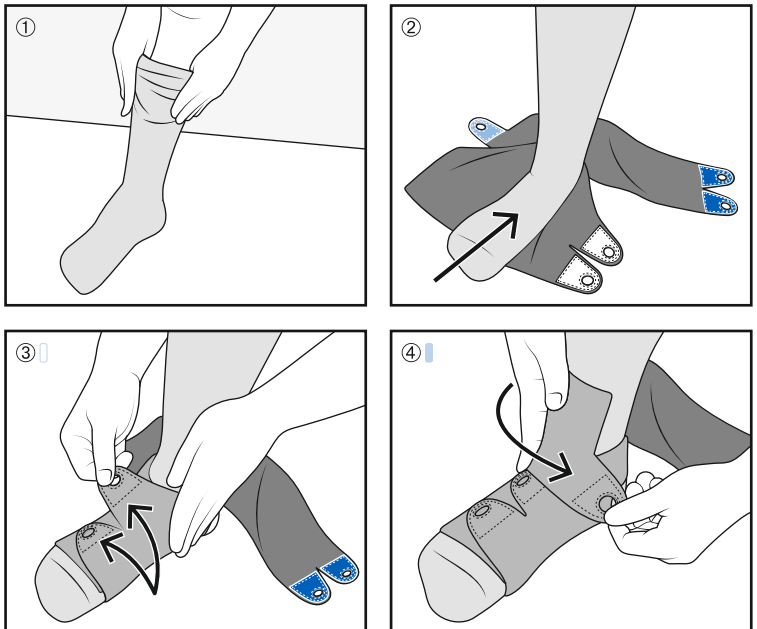
# ReadyWrap® Foot CT & Liner

Adaptives Kompressionssystem und Unterstrumpf  
Système de compression réglable et sous-bas  
Adjustable compression wrap and Liner  
Sistema compresivo ajustable y media interna  
Sistema compressivo regolabile e sottocalza  
Verstelbaar compressiesysteem en onderkous  
Justerbart kompressionssystem og inderstrømpe  
Justerbart kompressionssystem och inderstrumpa  
Säädettävä kompressiotuki ja alussukka  
Adaptivný kompresivný systém a spodná pančucha  
Áláltható kompressziós pántrendszer és alsóharisnya  
Podesivi kompresivni zavoj i donja čarapa  
Kohendatav kompressioontooded ja vooder  
Uyarlanabilir kompresyon sistemleri ve iç astar

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## en Instructions for Use

Thank you for choosing ReadyWrap®! ReadyWrap® is a low-stretch adjustable compression wrap designed for self-management of indicated conditions. ReadyWrap® Foot CT has adjustable straps to contour the foot and ankle providing the containment and compression needed for moderate to severe indications. All ReadyWrap® styles can be worn during the day and/or night. Please consult a trained medical professional prior to first use.

### Indications and patient groups

- Lymphoedema
- Lipoedema
- Oedema
- Venous ulcers
- Venous insufficiency
- Leg fatigue and/or tiredness

### Intended uses

- To maintain reduced fluid volume and shape of limbs
- Management of lymphoedema, lipooedema and oedema
- Relief of leg fatigue and heaviness
- Assist in management of some symptoms of venous ulcers
- Assist in helping limit venous ulcer recurrence
- Compression garments for phleological, lymphological indications (e.g., venous disease, chronic oedema and lymphoedema)

### Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

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### Contraindications

- Arterial insufficiency or ABPI < 0,8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (e.g., cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

### Precautions

- ABPI > 1,3
- Absent or impaired sensation (peripheral neuropathy)
- Any open wounds should be dressed prior to application
- Cardiac overload is suspected
- Patients have diabetes
- Patients have advanced small vessel disease
- Arterial disease is present
- Renal failure is present
- Rheumatoid arthritis is present

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## Donning instructions

Apply ReadyWrap® garments in the following sequence:

- 1 toe
- 2 foot
- 3 calf
- 4 thigh
- 5 knee

ReadyWrap® Foot CT garments are left/right-specific.  
ReadyWrap® Foot CT straps have built-in finger grips to make it easier to pull the straps while donning.

- 1 Slide the ReadyWrap® Liner onto your leg.
- 2 Unfasten all garment straps. With the colored hook facing up, place your foot inside the garment, aligning your heel in the heel opening (long strap with blue hook will be behind the heel).
- 3 Place the strap closest to your toes without the hook against the top of your foot. While holding that strap down, pull the opposite strap with the two-pronged white hook over the top at full stretch and fasten it on top of the strap below. Adjust for fit and comfort.
- 4 Pull the ankle strap with the light blue hook at full stretch across the front of your foot at a downward angle and fasten it to the opposite side. Adjust for fit and comfort.
- 5 Pull the ankle strap with the two-pronged white hook at full stretch around the front of your foot and fasten it to the opposite side at a downward angle or aligned with the strap below. Adjust for fit and comfort.

## Warranty

We warrant your ReadyWrap® garment to be free of material and manufacturing defects during the recommended six-month life of the garment.

## Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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**Care instructions**  
L&R recommends all ReadyWrap® garments be washed and dried before the first extended use.

- Wrap & Liner
- Machine wash cold (30°C), gentle cycle
  - Do not use fabric softener or bleach
  - Do not dry clean
  - Lay on a towel and squeeze your ReadyWrap® to remove excess water, lay flat on a dry towel to finish drying
  - Be sure that the VELCRO® brand fasteners do not contact your towel
  - Do not iron

## Material content

Wrap: Nylon (polyamide), polyurethane, spandex (elastane)  
Liner: Nylon (polyamide), spandex (elastane)

Does not contain latex: Natural rubber latex is not an intended ingredient in the production or packaging of the above products. We cannot completely exclude any potential contamination with traces of latex from production or the environment.

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VELCRO® is a registered trademark of Velcro BVBA.