Cellacare® Thorax Classic

Rippenbruchbandage für Frauen / Männer
Bande thoracique pour femme / homme
Thoracic support for women / men
Soporte torácico para mujeres / hombres
Bendaggio toracico per anatomia femminile / maschile
Thoraxbandage voor dames / heren

einzelner Patient – mehrfach anwendbar · un seul patient – à usage multiple
single patient – multiple use · un solo paciente – uso múltiple
singolo paziente – uso multiplo · één patiënt – meervoudig gebruik

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en Instructions for Use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics
Cellacare® Thorax Classic is an orthosis for stabilisation of the thorax area. A panel, 2 flexible stays and an elastic fabric adjustable with hook-and-loop fastener provide adaptable support.
- A version for women, height 15 cm with anatomical shape for the breast area
- A version for men, height 24 cm with straight fit

Product composition
Cotton, elastane, polyamide, polyester, polyethylene, polyurethane

Intended purpose
Support for compression and stabilisation of the thorax.

Indications
• after thoracic surgery
• rib fractures
• conservative treatment of thoracic contusions

Contra-indications
• Known allergy and / or hypersensitivity to any of the product components

Side effects
In very rare cases, skin irritations and / or allergies may occur.

Warnings and precautions
• Do not use Cellacare® Thorax Classic unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
• The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
• Unless specified otherwise by the health care professional Cellacare® Thorax Classic must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
• Cellacare® Thorax Classic shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.
• In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
• In case of pregnancy, the patient's treating physician needs to give the approval before using the product.
• Apply the product in accordance with the instructions for use. Please check the positioning of the product during use and correct as necessary.
• In case of discomfort or problems, consult a health care professional.

• If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
• Cellacare® Thorax Classic is intended for use on a single patient only.
• The product's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.
• Cellacare® Thorax Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).
• If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information
1. Apply the orthosis around the thorax.
2. Close the orthosis at the front by using the hook-and-loop fastener to obtain moderate compression.

Care recommendations
Machine wash the orthosis at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® Thorax Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Fasten the hook-and-loop closures and if possible use a mesh bag. Clean the hook-and-loop closures regularly so that they retain their ability to fasten tightly, and to preserve the performance of the product.

Storage and transport conditions
Cellacare® Thorax Classic must be kept away from light and moisture (preferably in the original packaging).

Disposal
In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information
In case of a serious incident, contact the manufacturer and the responsible health authorities.

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