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en Instructions for Use

Thank you for choosing ReadyWrap®! ReadyWrap® is a low-stretch adjustable compression wrap designed for self-management of indicated conditions. ReadyWrap® Knees has adjustable straps with a built in 50% overlap to contain the knee, while allowing movement. All ReadyWrap® styles can be worn during the day and/or night. Please consult a trained medical professional prior to first use.

Indications and patient groups

- Lymphoedema
- Lipoedema
- Oedema
- Venous ulcers
- Venous insufficiency
- Leg fatigue and/or tiredness

Intended uses

- To maintain reduced fluid volume and shape of limbs
- Management of lymphoedema, lipoedema and oedema
- Relief of leg fatigue and heaviness
- Assist in management of some symptoms of venous ulcers
- Assist in helping limit venous ulcer recurrence
- Compression garments for phlebological, lymphological indications (e.g., venous disease, chronic oedema and lymphoedema)

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (e.g., cellulitis)

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- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

Precautions

- ABPI > 1.3
- Absent or impaired sensation (peripheral neuropathy)
- Any open wounds should be dressed prior to application
- Cardiac overload is suspected
- Patients have diabetes
- Patients have advanced small vessel disease
- Arterial disease is present
- Renal failure is present
- Rheumatoid arthritis is present

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Donning instructions

- Apply ReadyWrap® garments in the following sequence:
- 1 toe
 - 2 foot
 - 3 calf
 - 4 thigh
 - 5 knee

1 Slide the ReadyWrap® Liner onto your leg.

- 2 Unfasten all garment straps.
- 3 Align the middle strap set at the back of knee with the hook side facing toward your knee.
- 4 Place and hold the middle strap without the hook (A) directly over your knee. Pull the opposing strap with the white hook (B) at

- 1 full stretch and fasten it directly on top of the strap below. (Strap set 1)
- 2 Fasten the strap set with the short horizontal light blue hook (C) below the knee at full stretch and fasten to the straps below. Next, pull the opposing strap with the long vertical light blue hook (D) at full stretch and fasten it directly on top of the strap below. (Strap set 2)
- 3 Fasten the strap set with the dark blue hook (E, F) above the knee using the same process. (Strap set 3)

Care instructions

L&R recommends all ReadyWrap® garments to be washed and dried before the first extended use.
Wrap & Liner
• Machine wash cold (30°C), gentle cycle
• Do not use fabric softener or bleach
• Do not dry clean
• Lay on a towel and squeeze your ReadyWrap® to remove excess water, lay flat on a dry towel to finish drying
• Be sure that the VELCRO® brand fasteners do not contact your towel
• Do not iron

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- Material content**
Wrap: Nylon (polyamide), polyurethane, spandex (elastane)
Liner: Nylon (polyamide), spandex (elastane)
- Does not contain latex:** Natural rubber latex is not an intended ingredient in the production or packaging of the above products. We cannot completely exclude any potential contamination with traces of latex from production or the environment.

Warranty

We guarantee your ReadyWrap® garment to be free of material and manufacturing defects during the recommended six-month life of the garment.

Disposal

Observe the current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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