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en Instructions for Use

Product description

Suprasorb® P PU-Schaumverband, nicht klebend, ist eine triple-layered wound dressing consisting an absorbent polyurethane foam between a perforated non-adherent wound contact layer and an outer waterproof film layer which is highly permeable to allow effective vapor transfer. Suprasorb® P non-adhesive PU Foam Dressing can effectively absorb wound exudates and maintain a moist wound environment.

Product composition
Polyurethane and Acrylic Adhesive

Intended purpose
The Suprasorb® P PU Foam Dressing can be used to treat all wounds, provided they are

-  moderately to heavily exuding and
-  superficial to
-  deep* wounds.

Indications

- venous leg ulcers
- pressure ulcers
- diabetic foot ulcers
- donor sites
- abrasions
- surgical wounds

Kontraindications
Do not use the product on patients with known sensitivity to Acrylic Adhesive or Polyurethane.

Application instructions
May be applied by lay persons under supervision of health care professionals.
The product can be used under compression bandaging.

1.) Preparing the wound
Clean the wound in accordance with current clinical standards.

2.) Applying the wound dressing
a) Select an appropriate dressing so that the foam pad overlaps the wound margins by approximately 2cm. Cut the dressing to size if necessary.
b) Place the foam side directly to the wound surface.
c) Secure with secondary dressing, such as bandage, adhesive tape or other appropriate secondary dressing.

*Where necessary, for deep wounds apply a packing rope as primary dressing (e.g. Alginates, hydrofibres and biocellulose).

3.) Dressing change
a) remove the retention dressing or bandage
b) lift the Suprasorb® P non-adhesive PU Foam Dressing away from the wound

Suprasorb® P non-adhesive PU Foam Dressings have the capacity to stay in place up to 7 days or until the exudate is visible and approaches 2cm from the edge of the foam pad, whichever is sooner. During the early stage of wound management, this product should be inspected frequently.

Precautions and warnings
1. If you are unsure of the severity of the wound or condition, consult a healthcare professional prior to the first use of this product.
2. In case of signs of infection (e.g. excessive pain or swelling), consult a healthcare professional for adequate infection treatment.
3. If reddening or sensitization occur, discontinue use.
4. Do not use this product with oxidizing agents such as hydrogen peroxide or hypochlorite solutions.
5. This product is a single use product. Opening the dressing pack compromises the sterile barrier therefore any unused dressing should not be retained for use at a later date.
6. Single product cannot be used on wound for more than 7 days.

Note
If any serious incident has occurred in relation to the use of Suprasorb® P non-adhesive PU Foam Dressing, it should be reported to the manufacture and competent authority of the Member State.

Storage and disposal
1. Please store the product in a clean and dry place in specified temperature range and away from sunlight.
2. Disposal should be handled according to local environmental procedures.

Shelf life
3 years from date of manufacture

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