
Cellacare® Genu Classic

Kniegelenkbandage
Bandage de l'articulation du genou
Knee support
Soporte para la articulación de la rodilla
Bendaggio per l'articolazione del ginocchio
Kniebandage
Bandáž pro kolenní kloub

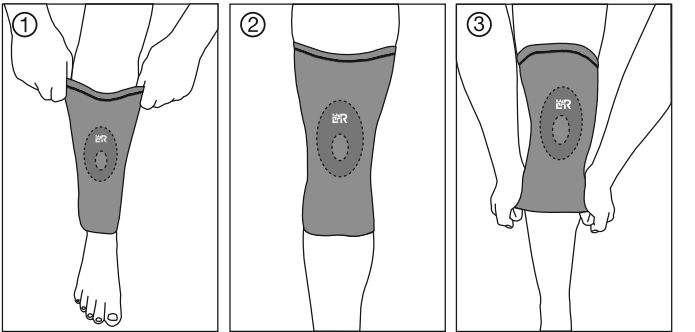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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:
Cellacare® Genu Classic is a knitted support with a compression effect which can improve proprioception. The integrated ring insert provides an intermittent massaging effect during movement and distributes the pressure. The support has two integrated lateral stays and can be worn on the right or left knee.

Product composition:
Elastodiene (natural rubber latex), cotton, polyamide, elastane, polyurethane, silicone, steel, polyester

Intended purpose:
Support for compression and stabilisation of the knee

Indications:
Treatment of acute or chronic pain and / or irritation of the knee joint, as well as functional deficits after injury or surgery and as a result of incorrect loading or overloading of the knee joint, capsule, tendons, muscles and / or ligaments in:

- mild to moderate knee sprains or contusions
- mild to moderate gonarthrosis or gonarthritits with and without joint effusion
- mild medial or lateral ligament instability
- patellofemoral pain syndrome
- mild patellar instability
- mild medial or lateral meniscal pathologies (acute traumatic or degenerative)
- postoperative treatment after meniscus surgery

Contra-indications:

- Known allergy and/or hypersensitivity to any of the product components

Side effects:
In very rare cases, skin irritations and/or allergies may occur.

Warnings and precautions:

- Do not use Cellacare® Genu Classic unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Genu Classic shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before application of the product.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of circulatory and / or sensory disorders (e.g. with diabetes mellitus); arterial diseases (occlusive arterial disease); venous disease (phlebitis, venous thrombosis), impaired lymph circulation in the legs please contact your treating physician / seek medical advice before application of the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- Cellacare® Genu Classic must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Genu Classic is intended for use on a single patient only.
- The product's durability may be compromised by certain factors, e.g. objects with sharp edges.
- Cellacare® Genu Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).

- This product contains natural rubber latex, which may cause allergic reactions.
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information:

Usage:

1. Grasp the right and left top of the support (top = end with product label) (Fig. 1).
2. Pull Cellacare® Genu Classic up over the knee so that the ring insert encircles the patella (Fig. 2).
3. To remove the support, grasp it by the lateral stays and pull it downwards and off over the foot (Fig. 3).

Care recommendations:
Wash the support at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® Genu Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. If possible use a mesh bag.

Storage and transport conditions:
Cellacare® Genu Classic must be kept away from light and moisture (preferably in the original packaging).

Disposal:
In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:
In case of a serious incident, contact the manufacturer and the responsible health authorities.

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