Suprasorb® CNP endo Kit AddOn

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en Instructions for Use

Table of Contents

- 1) Kit description and composition
- 2) Intended purpose
- 3) Indications
- 4) Contraindications
- 5) Side effects
- 6) Interactions
- 7) Storage
- 8) Preparation

- 9) Insertion of the additional foam drain in the wound cavity
- 10) Removal of the additional foam drain
- 11) Important notes / precautions
- 12) Disposal
- 13) Reporting obligation

1) Kit description and composition

The **Suprasorb® CNP endo Kit AddOn** consists of the following components:

- 1 foam drain (drain)** made of an open-pore foam (polyurethane, 20 mm diameter; 100 mm length) and a transparent drainage tube (polyurethane, 2.8 mm inner diameter and 4.4 mm outer diameter, 120 cm length) with proximal retention seam and distal loop
- 1 Y-connector** (polypropylene, incl. tube piece made of polyurethane)
- 1 retention dressing** (nylon, polyurethane, polyethylene, medical adhesive) 135 x 40 mm

The Suprasorb® CNP endo therapy unit with a connected exudate chamber is already available for generation of negative pressure.

2) Intended purpose

The Suprasorb® CNP endo Kit Add On is for use in adults to facilitate placement of two foam drains for negative pressure therapy in the rectum in conjunction with the Suprasorb CNP endo Kit Rectal, in combination with the Suprasorb® CNP endo therapy unit.

3) Indications

The indications of the Suprasorb® CNP endo Kit Rectal apply.

4) Contraindications

The contraindications of the Suprasorb® CNP endo Kit Rectal apply.

5) Side effects

The side effects of the Suprasorb® CNP endo Kit Rectal apply.

6) Interactions

There are no known interactions to date.

7) Storage

Store the Suprasorb® CNP endo Kit AddOn in a dry place at a maximum of 25°C and protected from dust and sunlight.

8) Preparation

Endoscopically confirm that Suprasorb® CNP endo therapy is indicated before starting the therapy. When determining the location, size and form of the defect that you would like to treat, you have also opted touse the foam drain of the Suprasorb® CNP endo Kit AddOn in addition to the foam drain of the Suprasorb® CNP endo Kit Rectal. Consult the Instructions for Use for the Suprasorb® CNP endo Kit Rectal. If required, trim the foam according to these Instructions for Use as well.

9) Insertion of the additional foam drain in the wound cavity

- a) First, insert the gelled foam drain of the Suprasorb® CNP endo Kit Rectal. When doing so, proceed according to the application steps 1, 2, 3 and 5 for intracavitary CNP endo therapy (“Placement of the proctoscope”, “Grasping the foam drain”, “Insertion of the foam drain into the rectum”, “Placement of the foam drain in the wound cavity”) in Chapter 10b of the Instructions for Use. The proctoscope remains in place.

- b) Then, insert the gelled foam drain of the Suprasorb® CNP endo Kit AddOn by releasing the grasper from the first foam drain, bringing it out of the patient and reinserting the grasped foam drain of the Suprasorb® CNP endo Kit AddOn through the proctoscope.
- c) Now place the foam drain of the Suprasorb® CNP endo Kit AddOn – adjusted to the geometry of the defect – beside the already placed foam drain of the Suprasorb® CNP endo Kit Rectal.
- d) Then, proceed according to application step 4 (“Removal of the proctoscope”) of Chapter 10b of the Instructions for Use for the Suprasorb® CNP endo Kit Rectal. Pull the two drainage tubes out of the proctoscope and connect them to the Y-connector of the Suprasorb® CNP endo Kit AddOn and then to the Suprasorb® CNP endo therapy unit.
- e) Application step 6 (“Starting the therapy”) follows.
- f) Only remove the endoscope after checking the position of the foam drain a final time. The x-ray detectability of the drainage tube allows the subsequent use of x-ray to check the positioning.
- g) Fix the drainage tubes with the aid of the supplied retention dressing as described in Chapter 10b.7 of the Instructions for Use for the Suprasorb® CNP endo Kit Rectal.

10) Removal of the additional foam drain

Remove the foam drain according to the application steps listed in Chapter 11) in the Instructions for Use for the Suprasorb® CNP endo Kit Rectal.

11) Important notes / precautions

- Training is required to use this Suprasorb® CNP endo Kit. You can receive this training from qualified personnel at L&R or through an authorised L&R distribution partner.
- Before using the Kit, ensure that the components are complete and intact.
- In the implementation of CNP endo therapy, the components of the Suprasorb® CNP endo Kit may only be replaced by components of other Suprasorb® CNP endo Kits.
- The implementation of the endoscopic procedure must be assessed by the respective healthcare professional responsible for the therapy, taking into consideration the benefits/risks for the patient.
- The CNP endo therapy may only be performed by physicians who are experienced in interventional flexible endoscopy of the gastrointestinal tract.
- The proctoscope, endoscope and foam drain must be inserted and removed carefully and never suddenly or with excessive force.
- If a severe bleeding complication is suspected (e.g. in case of abnormal drainage secretions), immediately perform endoscopic monitoring of the therapy under surgical conditions and in readiness to perform surgery.
- The user decides when the therapy is to be ended depending on the progress of wound healing. We recommend replacing the foam after therapy intervals of 48 to no more than 72 hours.
- Perform early endoscopic monitoring of the therapy in case of suspected therapy failure, for example, due to displacement or obstruction of the foam.

12) Disposal

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance - AVV). Recyclable packaging should be recycled according to the relevant national recycling systems.

13) Reporting obligation

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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