
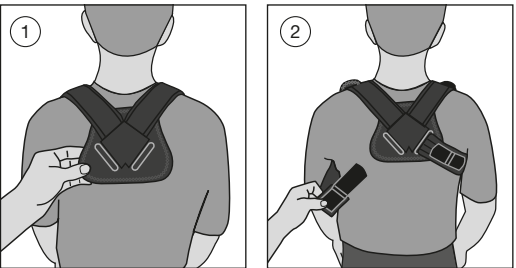
Cellacare® Clavicula Classic

Claviculabandage
Bandage de la clavicule
Clavicle Support
Vendaje para clavícula
Bendaggio per la clavicola
Claviculabandage
Bandáž klíční kosti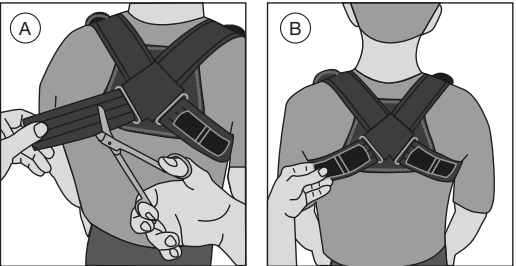

 einzelner Patient – mehrfach anwendbar · un seul patient – à usage multiple · single patient – multiple use · un solo paciente – uso múltiple · singolo paziente – uso multiplo · een patiënt – meervoudig gebruik · určeno pro jednoho pacienta – možno použiť opakovaně

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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:
Cellacare® Clavicula Classic is a support for Immobilisation of the shoulder according to the principle of the classical figure-of-eight bandage design.

Product composition:
Polyester, polyurethane, polyamide.

Intended purpose:
Support for the partial immobilisation and extension of the clavicle.

Indications:
• for the conservative treatment of clavicle (collarbone) fractures
• for postoperative and posttraumatic treatment

Contraindications:
• skin diseases and/or injuries in the application area (open, non-intact or injured skin)

Seek medical advice before applying/wearing Cellacare® Clavicula Classic in the following cases:
• allergic reactions to one or more of the components

Side effects:
Skin irritations and allergies can occur in very rare cases.

Usage information:
Adjusting Cellacare® Clavicula Classic needs to be done by trained and qualified personnel. In order to insure the required effect and a good fit, the shoulder straps may be shortened to the required length (Fig. A). The special 'Y' hook-and-loop fasteners can be removed and re-positioned to do this. The straps are the correct length when the 'Y' hook-and-loop fasteners can be drawn completely through the buckle while closing the straps (Fig. B).

- Apply Cellacare® Clavicula Classic while standing. Apply the support with the help of another person. When used for the first time, trained and qualified personnel need to demonstrate the correct application.
- Place the bandage over the shoulders insuring that the two permanently connected straps go over the shoulder to the front and the back plate is positioned between the shoulder blades (Fig. 1).
- Pass the straps under the arms to the back. Then pass the straps through the corresponding buckles, pull them tight and close them using the hook-and-loop fastener (Fig 2).

Warnings and precautions:
• Do not use Cellacare® Clavicula Classic unless directed by a health care professional. The first time patients use Cellacare® Clavicula Classic they need to have a health care professional show them how to put it on correctly.

- Apply Cellacare® Clavicula Classic in accordance with the instructions for use and illustrations provided. Please check the positioning of Cellacare® Clavicula Classic during use and correct as necessary.
- The health care professional will determine how long Cellacare® Clavicula Classic is to be worn per day based on the indications and healing progress.
- In case of discomfort or problems consult a health care professional.
- Cellacare® Clavicula Classic shall only be used on intact skin.
- Wounds have to be covered with a suitable wound dressing before use.
- Cellacare® Clavicula Classic is intended for use on a single patient only.
- Cellacare® Clavicula Classic's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fasteners. To prevent this, the hook-and-loop fasteners should always be fastened, for example when Cellacare® Clavicula Classic is not being worn or when being washed.
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Carre recommendations:
Machine wash at 30 °C (delicates cycle) with like colours. Do not bleach. Allow Cellacare® Clavicula Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Do not use fabric conditioner. Close the hook-and-loop fasteners and if possible use a mesh laundry bag. Clean the hook-and-loop fasteners regularly so they retain their ability to fasten tightly, and to preserve the performance of the product.

Storage and transport conditions:
Cellacare® Clavicula Classic must be kept away from light and moisture (preferably in the original packaging). Cellacare® Clavicula Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).

Disposal:
In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:
In case of a serious incident, contact the manufacturer and the responsible health authorities.

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Cellacare® is a registered trademark of Lohmann & Rauscher International GmbH & Co. KG