Cellacare® Abdominal Classic

Legal manufacturer
Intex s.a.
Pº Lizarkola s/n
20140 ANDOAIN (Guipúzcoa), Spain

Distributor / Distributeur
Lohmann & Rauscher International GmbH & Co. KG
Westerwaldstraße 4 • 56579 Rengsdorf, Germany
www.Lohmann-Rauscher.com

4039264/2020-03

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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:
Cellacare® Abdominal Classic is a support for stabilisation of the abdominal area

Product composition:
Cotton, elastane

Intended purpose:
Support and protection of the abdominal wall following surgery in the abdominal area

Indication:
Impairment of the abdominal wall following abdominal surgery

Contraindications:
• Known allergy and/or hypersensitivity to any of the product components

Side effects:
In very rare cases, skin irritations and/or allergies may occur.

Usage information:
A wide hook-and-loop fastener facilitates accurate fitting and adjustment of compression.

Warnings and precautions:
• Do not use Cellacare® Abdominal Classic unless directed by a health care professional. The first time patients use Cellacare® Abdominal Classic they need to have a health care professional show them how to put it on correctly.
• Apply Cellacare® Abdominal Classic in accordance with the instructions for use. Please check the positioning of Cellacare® Abdominal Classic during use and correct as necessary.
• The health care professional will determine how long Cellacare® Abdominal Classic is to be worn per day based on the indications and healing progress.
• In case of discomfort or problems consult a health care professional.
• Cellacare® Abdominal Classic shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.

- Cellacare® Abdominal Classic is intended for use on a single patient only.
- Cellacare® Abdominal Classic's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fasteners. To prevent this, the hook-and-loop fasteners should always be fastened, for example when Cellacare® Abdominal Classic is not being worn or when being washed.
- Cellacare® Abdominal Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).

Care recommendations:
Machine wash at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® Abdominal Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean, do not use fabric conditioner. Close the hook-and-loop fasteners and if possible use a mesh laundry bag. Clean the hook-and-loop fasteners regularly so it retains its ability to fasten tightly, and to preserve the performance of the product.

Storage and transport conditions:
Cellacare® Abdominal Classic must be kept away from light and moisture (preferably in the original packaging).

Disposal:
In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:
In case of a serious incident, contact the manufacturer and the responsible health authorities.

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