
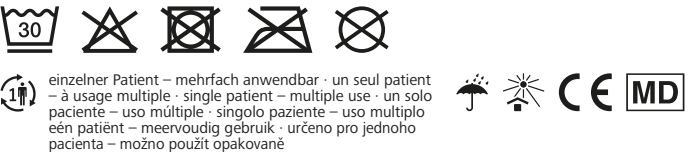
Cellacare®

Genucast Modular Classic

Modulare Knieschiene
Attelle modulaire de genou
Modular knee splint
Férula modular de rodilla
Stecca per il ginocchio modulare
Modulaire kniespalk
Modulární kolenní dlaha

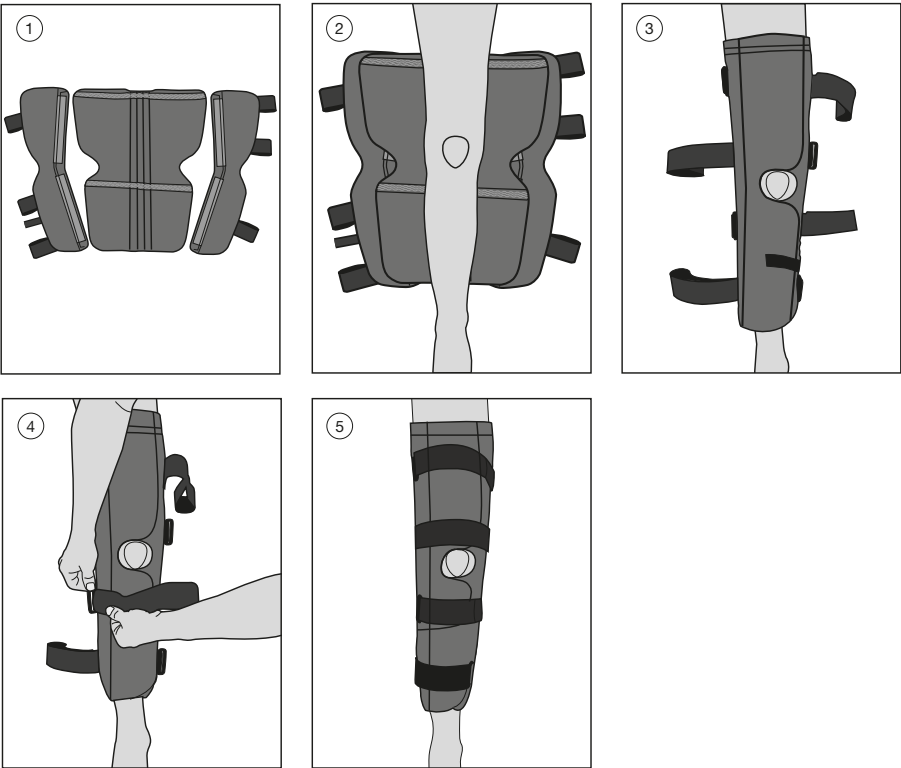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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:

Cellacare® Genucast Modular Classic is a three-element knee orthosis for immobilisation. It consists of one main panel and two side panels. Three-element structure allows for individual adaptation to the leg circumference. Conformable aluminium stays contribute to immobilisation of the knee joint.

Product composition:

polyester, polyurethane, polyoxymethylene, aluminium, polyamide, silica.

Intended purpose:

Orthosis for immobilisation of the knee joint

Indications:

- temporary immobilisation of the knee joint after injuries / operations

Contraindications:

- skin diseases and/or injuries in the application area (open, non-intact or injured skin)

Seek medical advice before applying/wearing Cellacare® Genucast Modular Classic in the following cases:

- allergic reactions to one or more of the components

Side effects:

Skin irritations and allergies can occur in very rare cases.

Usage information:

Adjusting Cellacare® Genucast Modular Classic needs to be done by trained and qualified personnel. Adjustments must be done in a lying position. Remove both side panels from the main panel (Fig. 1). Depending on the doctor’s prescription, adjust the angle of the rear aluminium stays, which adjust the knee flexion. If necessary, adjust the lateral aluminium stays to the patient’s leg circumference to ensure the required effect and good positioning. Wrap the main panel of the orthosis around the leg and re-attach both side panels to the main panel, the side panels should be adjusted to fit the patient’s leg circumference. Ensure the side panels are firmly secured to the main panel before fastening the orthosis.

The first time Cellacare® Genucast Modular Classic is used, trained and qualified personnel need to demonstrate how to put it on correctly.

1. The orthosis should be applied in a lying position. Place the orthosis under the leg so that the inner side with silicone bands is in contact with the leg. Make sure that the wider part of the orthosis is placed under the thigh and the narrow part is placed under the lower leg (Fig. 2).
2. Close the orthosis around the leg. Use the provisional fixation band. The front edges of the orthosis ideally overlap. Make sure that the patella is centered in the related opening (Fig. 3).
3. Guide the straps through the opposing buckles (Fig. 4).
4. To make sure the knee splint is positioned correctly, first tighten the straps directly above and below the knee. Afterwards tighten the other straps (Fig. 5).
5. The straps are adjusted correctly when the orthosis is in tight contact with the leg and when it does not constrict. If needed pull the straps tighter or loosen them.

Warnings and precautions:

- Do not use Cellacare® Genucast Modular Classic unless directed by a health care professional. The first time patients use Cellacare® Genucast Modular Classic, they need to have a health care professional show them how to put it on correctly.
- Apply Cellacare® Genucast Modular Classic in accordance with the instructions for use and illustrations provided. Please regularly check the positioning of the product during use and correct as necessary.
- The health care professional will determine how long Cellacare® Genucast Modular Classic is to be worn per day, based on the indications and healing progress.
- In case of discomfort or problems consult a health care professional.
- If local swelling or numbness occurs in the part of the body being treated, Cellacare® Genucast Modular Classic must be removed and a medically qualified health care professional consulted.
- Cellacare® Genucast Modular Classic should only be used on intact skin.
- Wounds must be covered with a suitable wound dressing before using Cellacare® Genucast Modular Classic.
- Cellacare® Genucast Modular Classic is intended for use on a single patient only.
- Cellacare® Genucast Modular Classic’s durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example, when the product is not being worn or when being washed.
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Care recommendations:

Remove all stays and clean them separately with a damp cloth. Pay attention to how the stays are orientated to ensure they are replaced in the correct way after washing. Wash the orthosis at 30 °C (delicates cycle) with like colours. Fasten all hook-and-loop closures. Do not bleach. Allow Cellacare® Genucast Modular Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Do not use any form of fabric conditioner. Clean the hook-and-loop closure regularly so it retains its ability to fasten securely, and to preserve the performance of Cellacare® Genucast Modular Classic.

Storage and transport conditions:

Cellacare® Genucast Modular Classic must be kept away from light and moisture (preferably in the original packaging). Cellacare® Genucast Modular Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).

Disposal:

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:

In case of a serious incident, contact the manufacturer and the responsible health authorities.

40525555/2020-11

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Cellacare® is a registered trademark of Lohmann & Rauscher International GmbH & Co. KG