Cellacare® Abdominal Classic

Abdominalbandage

Ceinture de soutien abdominal

Abdominal support belt

Soporte abdominal

Bendaggio addominale

Buikbandage

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en Instructions for Use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics

Cellacare® Abdominal Classic is an orthosis for stabilisation of the abdomen area. An abdominal panel, 2 flexible stays in the back and an elastic fabric adjustable with hook and loop fastener provide adaptable support.

Product composition

Cotton, elastane, polyamide, polyester, polyethylene, polyurethane

Intended purpose

Support for compression and stabilisation of the anterior abdominal wall

Indications

Supportive treatment in case of:

- postoperative and posttraumatic support of the abdomen (for example: after liposuction and abdominal plasty)
- weakness of connective tissue in the abdominal area
- regression after maternity

Contra-indications

- Known allergy and / or hypersensitivity to any of the product components

Side effects

In very rare cases, skin irritations and / or allergies may occur.

Warnings and precautions

- Do not use Cellacare® Abdominal Classic unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.

- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Unless specified otherwise by the health care professional, Cellacare® Abdominal Classic must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
- Cellacare® Abdominal Classic shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of pregnancy, the patient's treating physician needs to give the approval before using the product.
- Apply the product in accordance with the instructions for use. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Abdominal Classic is intended for use on a single patient only.
- The product's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.
- Cellacare® Abdominal Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information

1. Center the abdominal panel.
2. Fasten the orthosis by using the hook-and-loop fastener to obtain moderate compression.

Care recommendations

Machine wash the orthosis at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® Abdominal Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Fasten the hook-and-loop closures and if possible use a mesh bag. Clean the hook-and-loop closures regularly so that they retain their ability to fasten tightly, and to preserve the performance of the product.

Storage and transport conditions

Cellacare® Abdominal Classic must be kept away from light and moisture (preferably in the original packaging).

Disposal

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information

In case of a serious incident, contact the manufacturer and the responsible health authorities.

1 einzelner Patient – mehrfach anwendbar · un seul patient – à usage mutiple · single patient – multiple use · un solo paciente – uso múltiple · singolo paziente – uso multiplo · één patiënt – meervoudig gebruik

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