
Cellacare® Epi Classic

Ellenbogenbandage
Bandage de coude
Elbow support
Soporte para codo
Bendaggio per il gomito
Elleboogbandage
Bandáž pro loketní kloub

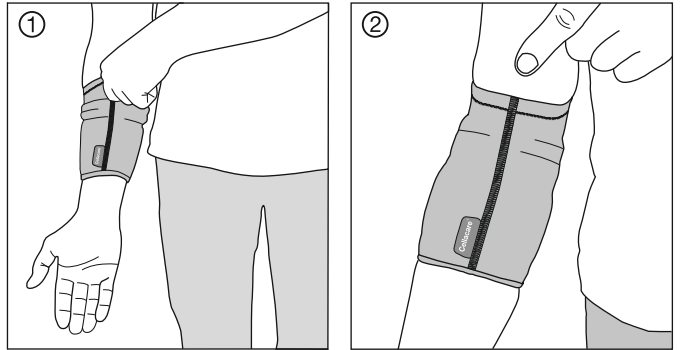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en Instructions for Use

Dear customer,

Thank you for choosing an L&R
product.

Please read the instructions for use
carefully. If you have any questions,
please contact your physician or
specialist shop.

Product description and performance characteristics:

Cellacare® Epi Classic is a knitted support
with a compression effect. Inserts on both
sides provide a massaging effect. The
support can be worn on the right or left
elbow.

Product composition:

Cotton, elastane, polyamide, polyester,
polyurethane, synthetic latex

Intended purpose:

Support for compression and relief of the
elbow and the associated muscle-tendon
units

Indications:

Treatment of acute or chronic pain with
functional deficits as a result of incorrect
loading or overloading of the extensor and
flexor musculature of the lower arm with
tendon irritation (tendinosis):

- Lateral epicondylitis (tennis elbow)
- Medial epicondylitis (golfer's elbow)

Contra-indications:

- Known allergy and / or hypersensitivity
to any of the product components

Side effects:

In very rare cases, skin irritations and / or
allergies may occur.

Warnings and precautions:

- Do not use Cellacare® Epi Classic unless
directed by a health care professional.
The first time patients use the product
they need to have a health care
professional show them how to put it
on correctly.

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- Cellacare® Epi Classic must not be
used beyond the last day of the month
indicated on the packaging (expiry date).
- If there are excessive signs of wear
that could influence the performance
of the product, consult a healthcare
professional and replace the product if
necessary.

Usage information:

Usage:

1. Grasp the top edge of Cellacare® Epi
Classic (green line), slip it over your
forearm and pull it up to the elbow joint
(Fig. 1).
2. Adjust Cellacare® Epi Classic so that the
inner longitudinal seam runs centrally
along the inside of the elbow (Fig. 2).

Care recommendations:

Wash the support at 30 °C (delicate cycle)
with like colours. Do not bleach. Allow
Cellacare® Epi Classic to air-dry away from

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heat sources; do not dry in a dryer. Do not
iron and do not dry clean. If possible use a
mesh bag.

Storage and transport conditions:

Cellacare® Epi Classic must be kept away
from light and moisture (preferably in the
original packaging).

Disposal:

In Europe, product waste can be assigned a
waste code from Chapter 18 01 and
packaging waste a waste code from
Chapter 15 01 of the Directive on the
European Waste Catalogue (Waste
Catalogue Ordinance – AVV). Packaging
labelled as recyclable should be taken to
the applicable national recycling systems.

General information:

In case of a serious incident, contact the
manufacturer and the responsible health
authorities.

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