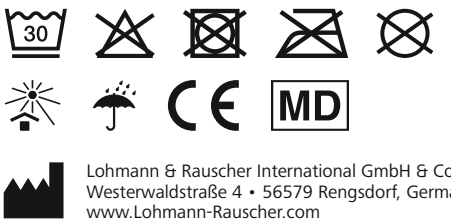
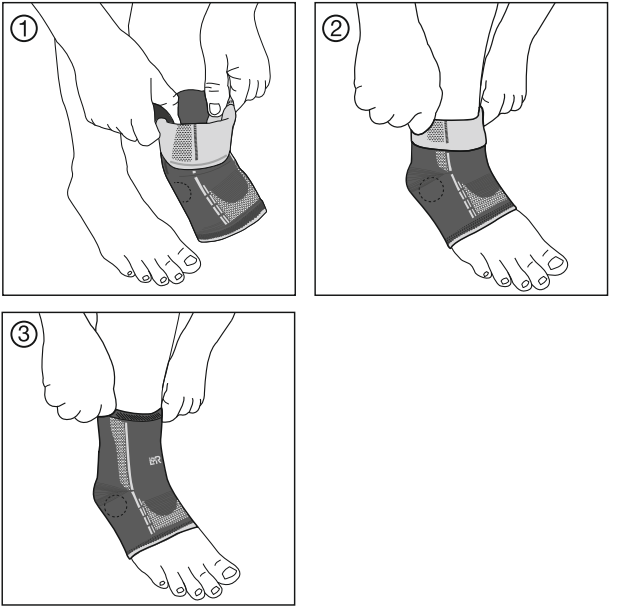
Sprunggelenkbandage
Bandage de la cheville
Ankle support
Soporte para el tobillo
Bendaggio per l'articolazione della caviglia
Enkelbandage
Bandáž na hlezenní kloub

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en Instructions for Use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:
Cellacare® Malleo Comfort is a knitted support with a compression effect which can improve proprioception. Silicone inserts on both sides provide a massaging effect. The support can be worn on the right or left ankle.

Product composition:
Elastane, polyamide, polyester, cotton, silicone, polyurethane

Intended purpose:
Support for compression and stabilisation of the ankle

- Indications:**
- Acute and chronic pain in the ankle joint
 - Ligament injuries to the upper ankle (grade 1)
 - Moderate chronic ligament instability
 - Moderate local hypermobility
 - Low grade osteoarthritis, activated osteoarthritis, arthritis
 - Perimalleolar swelling, joint effusion, intra-articular irritation

Contra-indications:

- Known allergy and / or hypersensitivity to any of the product components

Side effects:
In very rare cases, skin irritations and / or allergies may occur.

- Warnings and precautions:**
- Do not use Cellacare® Malleo Comfort unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
 - The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
 - Cellacare® Malleo Comfort shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before application of the product.
 - In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
 - In case of circulatory and / or sensory disorders (e.g. with diabetes mellitus); arterial diseases (occlusive arterial disease); venous disease (phlebitis, venous thrombosis), impaired lymph circulation in the legs please contact your treating physician / seek medical advice before application of the product.
 - Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
 - In case of discomfort or problems, consult a health care professional.
 - Cellacare® Malleo Comfort must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
 - If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
 - Cellacare® Malleo Comfort is intended for use on a single patient only.
 - The product's durability may be compromised by certain factors, e.g. objects with sharp edges.
 - Cellacare® Malleo Comfort must not be used beyond the last day of the month indicated on the packaging (expiry date).
 - If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information:

- Usage:**
1. Fold over the top of the Cellacare® Malleo Comfort as far down as the silicone insert (Fig. 1).
 2. Grasp the Cellacare® Malleo Comfort in the middle of the silicone inserts and pull it over your foot until the heel part is firmly in place (Fig. 2).
 3. Unfold the top of the Cellacare® Malleo Comfort and pull it upwards, if necessary (Fig. 3). Ensure that the silicone inserts are sitting correctly on both sides of the ankle.

Care recommendations:
Wash the support at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® Malleo Comfort to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. If possible use a mesh bag.

Storage and transport conditions:
Cellacare® Malleo Comfort must be kept away from light and moisture (preferably in the original packaging).

Disposal:
In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:
In case of a serious incident, contact the manufacturer and the responsible health authorities.

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