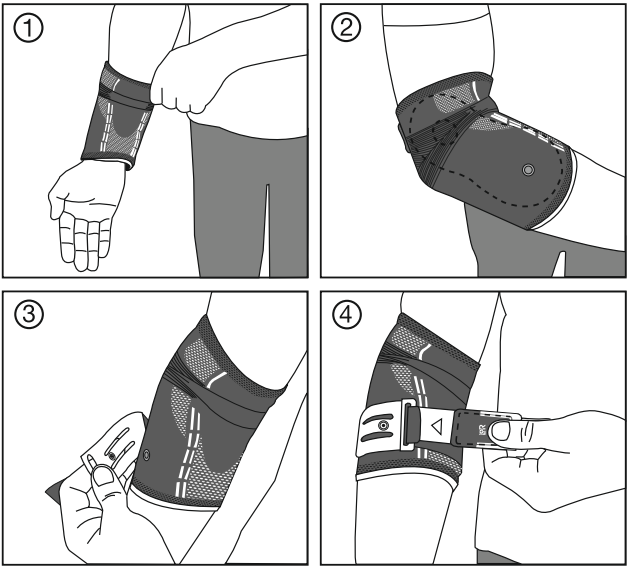
Cellacare®
Epi Comfort

Ellenbogenbandage
Bandage de coude
Elbow support
Soporte para codo
Bendaggio per il gomito
Elleboogbandage
Bandáž pro loketní kloub

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en Instructions for Use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:
Cellacare® Epi Comfort is an active support providing compression around the elbow joint and on the muscle-tendon units surrounding the joint. The removable strap adds extra compression directly where needed and can be adjusted if necessary.

Product composition:
Stainless steel, cotton, elastane, polyamide, polyester, polypropylene, polyurethane, thermoplastic polyurethane, silicone.

Intended purpose:
Support for compression and relief of the elbow and the associated muscle-tendon units

Indications:
Treatment of acute or chronic pain with functional deficits as a result of incorrect loading or overloading of the extensor and

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flexor musculature of the lower arm with tendon irritation (tendinosis):

- Lateral epicondylitis (tennis elbow)
- Medial epicondylitis (golfer's elbow)

Contra-indications:
Known allergy and/or hypersensitivity to any of the product components

Side effects:
In very rare cases, skin irritations and/or allergies may occur.

Warnings and precautions:
Do not use Cellacare® Epi Comfort unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.

- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Epi Comfort shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.

- In case of circulatory and / or sensory disturbances in the arms, disorders of the lymphatic system in the arms, phlebitis and thrombosis, please contact your treating physician / seek medical advice before application of the product.
- In case of specific morphological peculiarities of the elbow being treated which restrict or complicate treatment with the support and its correct application, please contact your treating physician / seek medical advice before application of the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- Cellacare® Epi Comfort must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Epi Comfort is intended for use on a single patient only.
- The product's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.
- Cellacare® Epi Comfort must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.


Usage information:

Usage:


1. Grasp the top edge of the support (Cellacare® label) and pull it over your forearm (Fig. 1).
2. Adjust the support so that the inside of the elbow is positioned in the flexible area of the knitted fabric.
3. The longitudinal external seam runs behind the elbow and the silicone inserts encircle the bony protuberances (interior and exterior) (Fig. 2).
4. If required, you can use the strap provided to increase the compression. It is affixed to the press stud on the outside of the support to bring it from the exterior to the interior of the forearm

(Fig. 3). The triangle on the fastener shows the amount of tension; proper tension is indicated by the shape of the triangle (Fig. 4 and pictograms). Take care not to pull the strap too tight.

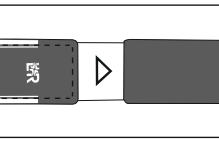
Insufficient tension

Correct tension

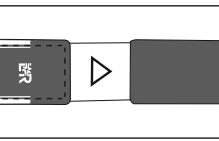Excessive tension

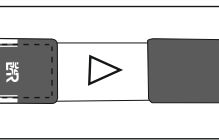Care recommendations:

Remove the stabilizing strap before washing (close the hook-and-loop fastener). Wash the support at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® Epi Comfort to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. If possible use a mesh bag. Clean the hook-and-loop closure and the press stud regularly to preserve the performance of the product.

Storage and transport conditions:
Cellacare® Epi Comfort must be kept away from light and moisture (preferably in the original packaging).

Disposal:

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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